
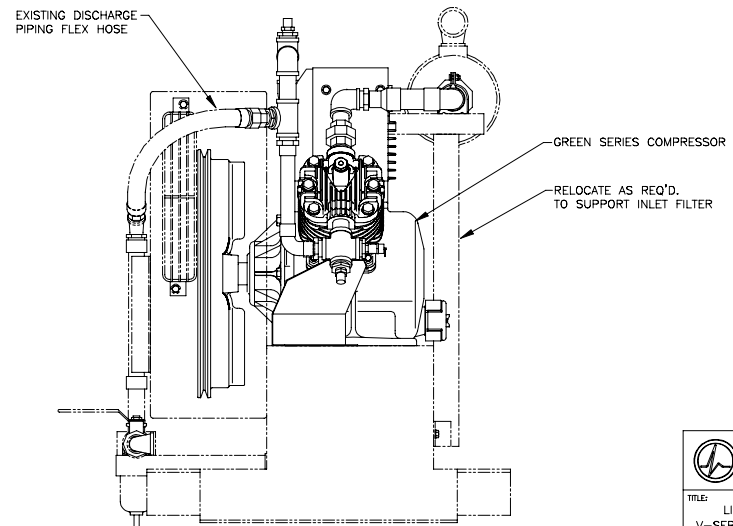


TOP VIEW - BASE
AIR DEFLECTOR MOUNTING HOLE LOCATIONS

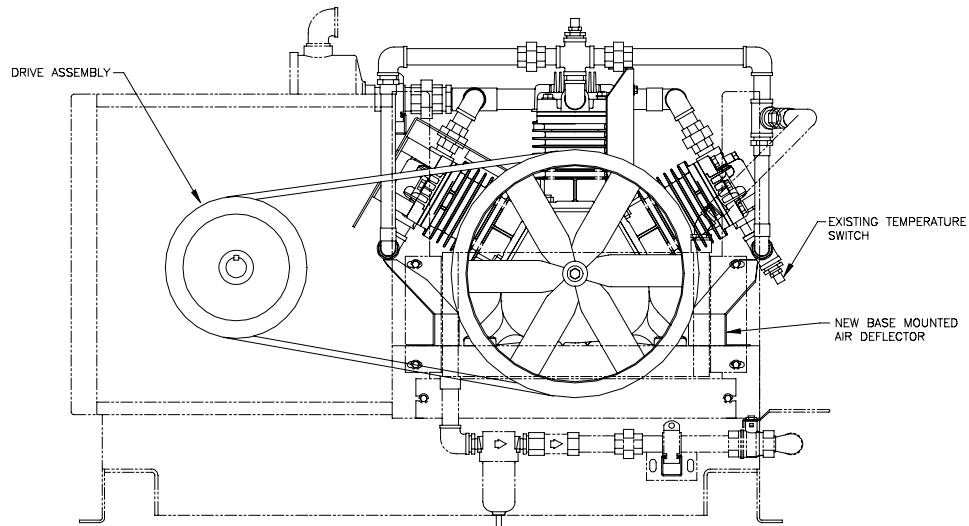

NOTES:

1. RETROFIT KIT CAN BE USED TO REPLACE ANY 10HP V-SERIES COMPRESSOR ON MODELS LTM, BTM, LMS AND LPS MEDICAL AIR UNITS.
2. ALL AIR INLET PIPING AND DISCHARGE PIPING TO BE ASSEMBLED IN THE FIELD BY A CERTIFIED BEACON-MEDÆS SERVICE TECHNICIAN.
3. COMPRESSOR SUPPLIED WITH BOTH AIR INLET AND AIR DISCHARGE MANIFOLD ASSEMBLIES FACTORY INSTALLED.
4. KIT DOES NOT INCLUDE: DISCHARGE PRESSURE RELIEF VALVE, HIGH AIR TEMPERATURE SWITCHES OR UNLOADER CONTROL PIPING. THESE ITEMS MUST BE TRANSFERRED FROM THE OLD COMPRESSOR TO THE NEW GREEN SERIES COMPRESSOR.
5. REPLACE EXISTING MOTOR SHEAVE AND DRIVE BELT WITH THE ONES SUPPLIED WITH THE RETROFIT KIT.
6. REMOVE AND DISCARD EXISTING AIR DEFLECTOR LOCATED ON THE COMPRESSOR BASE AND DRILL NEW HOLES AS SHOWN FOR NEW DEFLECTORS SUPPLIED WITH THE KIT.

		TITLE: LIFELINE 15HP DPX RETROFIT KIT V-SERIES TO GREEN SERIES COMPRESSOR	
		DRAWN BY: [blank] CHECKED BY: [blank]	DATE: [blank]
REVISED BY: [blank]	DATE: [blank]	DRAWING NO. DIA 03 075	REV. A
SCALE: 1:5	D	DIA 03 075	A